

NewGel™+ Case Studies

Reduction of Trauma-Induced Scar



Sutures after Trauma



After 4 months of NewGel™+ use

Second Degree Burn Scar Treatment



Before NewGel™+



After 2 months of NewGel™+ use

Reduction of Recurring Keloid Scar



Keloid prior to surgery



Recurring post-operative scar



After 8 weeks of NewGel™+ use




NewGel™+
SCAR MANAGEMENT SOLUTION

24hr Treatment
for **MAXIMUM Scar**
Reduction

Introducing NewGel™+E

Advanced Silicone Gel with Vitamin E



NewGel™+E ointment was developed for use when sheets would be inconvenient, hard to secure or unsightly

- Perfect for small scars and scars in areas requiring high flexibility
- Ideal for facial scars and post-laser treatment
- Contains Vitamin E to help soften and moisturise scars

For Optimal Treatment: While NewGel™+E ointment can be used alone, for best results, use it in conjunction with NewGel™+ sheets. Apply one product during the day and the other at night for 24 hour a day treatment.

Ask your physician about NewGel™+ today!

For More Information:

Call 0845 130 5866 or visit www.newgel.co.uk


ADVANTECH
SURGICAL

Advantech Surgical Ltd.

Southgate Chambers, 37-39 Southgate Street, Winchester,
Hampshire, SO23 9EH



CE & FDA approved for scar treatment

www.newgel.co.uk



24hr Treatment for MAXIMUM Scar Reduction

Safe, effective, easy to use

What makes NewGel™+ different?

- Made with medical grade silicone formulated to provide the best adhesion, durability and conformability
- NewGel™+ are full thickness sheets for optimal occlusion, hydration and pressure.
- NewGel™+ silicone gel sheets are available in two backings:
 - *Beige - thickest, most durable*
 - *Clear - where appearance is a consideration*
- NewGel™+ is CE marked and FDA approved for scar management.

How is NewGel™+ used?

- Peel off the release liner and apply directly to the scar.
- Wear for a minimum of 6 hours per day.
- Reusable for up to 4 weeks. Simply hand wash with water, rinse well, air dry and re-apply.



Do not use NewGel™+ on any open wound. Apply only to fully closed sites (sutures removed, no scabbing or oozing).

PRODUCT	ITEM NO.	ITEM DESCRIPTION
	NG-101	13cm x 15cm Sheet Beige (1 per box)
	NG-101S	2.5cm x 15cm Strips Beige (4 per box)
	NG-301	13cm x 15cm Sheet Clear (1 per box)
	NG-301S	2.5cm x 15cm Strips Clear (4 per box)
	NG-120	Areola Circles Beige (1 pair per box)
	NG-320	Areola Circles Clear (1 pair per box)
	NG-124	Lollypop Beige (1 pair per box)
	NG-324	Lollypop Clear (1 pair per box)
	NG-126	Anchor Beige (1 pair per box)
	NG-326	Anchor Clear (1 pair per box)
	NG-180	2.5cm Dot Beige (6 per tray)
	NG-380	2.5cm Dot Clear (6 per tray)
	NG-140	Ear Beige (1 pair per box)
	NG-340	Ear Clear (1 pair per box)
	NG-144	3cm x 2cm Beige (2 per tray)
	NG-344	3cm x 2cm Clear (2 per tray)
	NG-146	Nose Beige (1 pair per box)
	NG-346	Nose Clear (1 pair per box)
	NG-160	20cm Abdomen Beige (2 per box)
	NG-360	20cm Abdomen Clear (2 per box)
	NG-164	61cm Abdomen Beige (1 per box)
	NG-364	61cm Abdomen Clear (1 per box)
	NG-170	25cm x 61cm Large Blanket Beige
	NG-370	25cm x 61cm Large Blanket Clear
	NGO-800	15g NewGel™+E Advanced Silicone Gel

Clinically proven to reduce and flatten old and new scars

What is NewGel™+?

NewGel™+ is 100% medical grade silicone gel sheeting for managing, treating and reducing scars. It is self-adhesive and worn directly on the scar.

How does it work?

NewGel™+ reduces, flattens and smooths scars by providing a combination of occlusion (seals the scar from exposure to air), hydration, elevated skin temperature and continuous light pressure.

How soon will I see results?

Positive results can be seen in as little as 3 to 7 weeks. Best results are typically seen after 3 months.

What type of scar will NewGel™+ work on?

Effective on all types of scars, both old and new:

- Keloid
- Hypertrophic
- Trauma
- General surgical
- Burn
- Cosmetic procedures
- C-Section
- Abdominoplasty

Is NewGel™+ clinically proven?

Yes, there have been over one hundred clinical studies confirming silicone gel's effectiveness in treating and preventing scars.